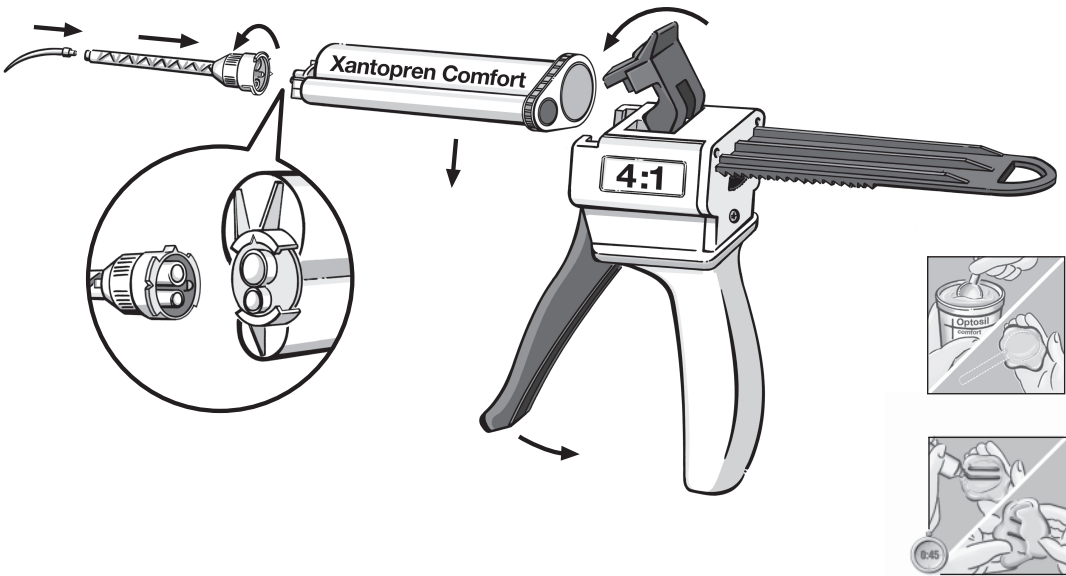
Technische Informationen

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Technical Information

Silicon based precision condensation curing impression material.

Optosil®- Xantopren®- Comfort®	EN ISO 4823	Colours	Dosage			Mixing- time	Working- time	Time in mouth	Shore Hardness	Maximum deformation under pressure	Recovery from deformation	Linear dimensional change
			Base	Activator								
				Liquid	Paste							
Optosil® Comfort®	Type 0	yellow	1 meas. spoon (10.7 g / 9.5 ml)	6 - 8 drops (0.14 g / 0.13 ml)	6 - 8 graduations (0.32 g / 0.29 ml)	30	1:00	4:00	58	2.5	98	0.2
Xantopren® Comfort® medium	Type 2	green	///	///	///	///	2:30	3:30	38	7.5	> 98	1.0
Xantopren® Comfort® light	Type 3	blue	///	///	///	///	2:00	3:00	36	8.0	> 98	1.0

Type 0 = putty consistency
Type 2 = medium bodied consistency
Type 3 = light bodied consistency

Graduation marks as indicated on the mixing pad
and mixing spatula (1 graduation = 1,3 cm)

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as and to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control, and therefore, entirely your own responsibility. Should, in spite of this, liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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User Advice

The **Optosil®/Xantopren®** assortment is a complete system.

Prior to first use extrude some material from the cartridge. After use replace mixing tip by protecting cap (hygienics).

For best results usage with other silicon materials and Activators is not recommended.

Pouring the model:

Pouring can take place 30 minutes after the impression is removed from the mouth. Impressions remain dimensional stable up to 7 days.

Recommended model material:

All commercially available modeling stones can be used.

Our suggestion:
Class III Gips, Moldano®/Moldadu®
Class IV Gips, Moldastone®/Moldasynt®

Electroplating:

Impressions may be copper or silver plated in the usual way.

Disinfection:

Impressions should generally be treated by immersion for 10 minutes* in 5,25 % Sodium Hypochlorite solution. Other silicon specific disinfectant solutions should be used as directed by the products manufacturer.

* Gelson Luis Adabo, DDS, PhD, Elaine Zaranotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD
Effect of disinfectant agents on dimensional stability of elastomeric impression materials. J Prosthet Dent, 81, 621-4, 1999

We always recommend the use of our Universal Adhesive.

Do not store above 25°C.

Adhesion:

Storage conditions:

Warning Activator:

Avoid contact with eyes. Observe the usual precaution measures recommended for the handling of chemicals. Wear suitable gloves.

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