

V-varnish™

Cavity varnish

CE
1023

Description

V-varnish™ is fluoride varnish containing 50mg of sodium fluoride per 1ml. It also contains TCP and xylitol. It is applied to teeth for the purpose of delivering fluoride and treatment hypersensitivity. It has four flavors such as bubble gum, strawberry, melon and mint. V-varnish™ is packed in single dose package allows easy mixing and application.

Composition

- Rosin
- Sodium Fluoride
- Ethanol
- Xylitol
- TCP

Indications

- Treatment of hypersensitive teeth
- Use on exposed dentin and root sensitivity
- Caries prophylaxis

Contra Indications

1. Do not use on patients with ulcerative gingivitis or stomatitis.
2. If patients are known to be allergic to ingredients including fluoride, rosin or similar materials, V-varnish™ should not be used.
3. After the application of V-varnish™, patients should avoid chewing hard food, drinking liquids containing alcohol and brushing for 4 to 6 hours.

Recommended Procedure

1. Thoroughly clean the teeth and remove excess moisture with cotton rolls and air.
2. Peel back the foil cover until the well containing V-varnish™ is exposed.
3. Mix V-varnish™ well using a brush thoroughly.
4. Apply V-varnish™ thinly with a brush.
5. It is recommended to remain V-varnish™ on the teeth surface as long as possible at least 4 hours.

Storage

1. Dispose the remained materials after using V-varnish™ because it is a single dose package.
2. Do not store at elevated temperature or intense light.

Precautions

1. It should be used by a dentist and dental professional. Observe the procedure and the precautions.
2. Do not use if the package is damaged.

3. Do not get other fluoride treatments on the same day. Also refrain from using fluoride supplements and rinses for several days after treatment.
4. Do not use after expiry date.
5. For a single use only.

Symbols



Use by




Lot number



Consult
Instructions for use



Do not reuse

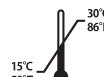

15°C
59°F
30°C
86°F
Temperature
limitation



Manufacturer



Caution



Authorized representative
in the European community

Warranty

VERICOM CO.,LTD will replace product that is proven to be defective. VERICOM CO.,LTD does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.



VERICOM CO., LTD.

861-8, Toegye-Dong, Chuncheon-Si,
Gangwon-Do, 200-944, Korea
Tel: +82-31-441-2881 Fax: +82-31-441-2883
E-mail: vericom@vericom.co.kr
www.vericom.co.kr

European Representative (Koo Kyo-Chan)

3 Rue Rosiers 92340 Bourg la Reine France
Tel: +33-1-4973-7930 Fax: +33-1-7402-5930