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Calciumhydroxide Paste 45%

Instructions for use

Patient target group
Children, adolescents, adults and older patients

Intended users
Dentist

Composition
Qualitative information according to the information in the biocompatibility:
Calcium hydroxide, water, barium sulphate, hydroxyethyl cellulose

Quantitative data:
Substances (Quantity according to biocompatibility):
45 % calcium hydroxide,
5 % barium sulphate

Intended purpose of the product
Calciumhydroxide Paste 45% is an aqueous calcium hydroxide paste.

Indications
Treatment of caries-free dentin close to the pulp, deep caries treatment, indirect pulp capping, direct pulp capping, pulpotomy, insertion in the root canal.

Contraindications
The use of Calciumhydroxide Paste 45% is contraindicated: in case of pulp opening/exposure in carious dentin, in case of inflammatory pulp changes, in case of hypersensitivity to one of the ingredients.

Performance characteristics of the product
Alkalisierung through calcium hydroxide

Specific precautions
None

Application
The amount of Calciumhydroxide Paste 45% to be administered depends on the respective application:
1. Deep caries treatment / indirect pulp capping - Calciumhydroxide Paste 45% is applied to the cavity floor as a thin film without pressure. A temporary or final filling is then placed.
2. Direct pulp capping - After the working area has been dried, Calciumhydroxide Paste 45% is applied and a suitable covering filling is then placed to seal the tooth permanently.
3. Vital amputation - Calciumhydroxide Paste 45% creates a favourable condition for healing of the remaining pulpal stumps due to the recognised localised corrosive effect of calcium hydroxide on the residual pulp.
4. Underfilling / pulp protection - The space-saving radiopaque Calciumhydroxide Paste 45% also forms a thin whitish insulating layer against cement acids or free monomer in the anterior region under silicate or resin fillings as pulp protection.

5. Dentin wound dressing for ground vital teeth - Calciumhydroxide Paste 45% is applied to the prepared tooth stump with a brush or cotton swab and carefully dried with warm air. Avoid contact with the gingiva. The brush should first be dipped in boiled water or distilled water to dilute the paste slightly.
6. Medical insertion into infected root canals. Each root canal should be filled with Calciumhydroxide Paste 45% without air bubbles. If necessary, the insert must be renewed after 6 to 8 weeks.

Warnings
Residual risks
Calciumhydroxide Paste 45% must always be kept tightly closed.
Do not apply increased pressure to the opened pulp during application. Calcium hydroxide causes local irritation and corrosive effects on skin, eyes and mucous membrane. Calcium hydroxide particles on the cornea and in contact with the eye must be removed immediately by suitable mechanical means, then rinse immediately and thoroughly with water. This medical device must not be used after the expiry date.
This medical device must be kept out of the reach of children.

Side effects
Adverse side effects are not known. Risks in case of overdose are not known.

Interactions
Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage
Calciumhydroxide Paste 45% should be stored between 15° C - 23°C.

Minimum shelf life
Do not use after the expiry date.

Variants
[\[REF\]](#)24120 1 x syringe with 2.0 ml paste + cannulas

Disposal
Unused material can be disposed of with household waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

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